## Xineos-1501/2301

CMOS X-ray Detectors for Frame-Based Extra-Oral Dental Scanning



#### **Key Features**

- Best in class low-dose image quality
- 99µm pixel pitch
- 300fps at full resolution
- 1,000 fps or more in 198 um ROI-mode
- Pixel saturation dose switchable
  through communications interface
- Frame based imaging for focal plane optimization (reconstructed 2D)
- Digital TDI mode for retrofitting to traditional CCD-TDI systems
- Stable offset calibration
- Low power dissipation
- Gigabit Ethernet data interface (LVDS option available on request)

#### **Typical Applications**

- Dental Panoramic
- Dental Cephalometric

# Xineos CMOS Scanning X-ray Detectors

The Xineos-1501 and Xineos-2301 CMOS detectors set a new benchmark in low dose imaging performance, which is critical for high frame rate scanning applications. Built with our sixth generation CMOS technology, these detectors offer seamlessly switchable saturation dose to maximize dynamic range or sensitivity on demand.

The Xineos-1501 detector delivers excellent image quality at high frame rates, supporting image sequence acquisition at high angular resolution to enable reconstructing Panoramic images with optimized focal planes through the patient-specific dental arch. Such functionality eliminates the risk of excess dose to the patient in case of scan trajectory misalignment.

The Xineos-1501 and Xineos 2301 also feature a novel integrated "Digital Time Delayed Integration" (or DTDI) acquisition mode, to simulate the image acquisition mode of traditional (analog) CCD-TDI scanning detector. In this mode, the detectors deliver signal-to-noise, sensitivity and dynamic range performance rivaling those of CCD-TDI detectors, and make it easy to retrofit these cost effective detectors to existing Dental Panoramic and Cephalometric systems that lack the processing power to support advanced frame based reconstruction software.

With only 6mm edge distance at the patient shoulder side, these detectors enable improved patient access and compact enclosure designs.


#### SPECIFICATIONS (TYPICAL VALUES @ RQA5)


Parameter Pixel Pitch Active Area Resolution Binning support Scintillator Seamlessly switchable saturation modes Saturation Dose, RQA5 (per mode) Dynamic Range (per mode) MTF @ 1lp/mm / 2lp/mm Non-linearity ADC Conversion Data Interface Frame Rate - Full size, full resolution - Full size, 2x2 binning **ROI** readout Trigger modes X-Ray Energy Range Power Consumption (active) Dimensions (WxHxD) Weight

Xineos-1501	Xineos-2301
99µm	99µm
152x6.5mm <sup>2</sup>	228x6.5mm <sup>2</sup>
1536x68	2304x68
1x1 / 2x2	1x1 / 2x2
Medical-grade colur	nnar Csl
2 modes, software switchable	
2.5uGy / 12uGy	2.5uGy / 12uGy
72dB / 76dB	72dB / 76dB
60% / 30%	60% / 30%
<1%	<1%
14-bit	14-bit
GigE	GigE
300fps	300fps
600fps	600fps
Programmable position & width	
Continuous or Synchronized	
40125kVp	
7W	
91x240x21mm	
0.6kg	




Xineos-1501/2301 CMOS X-ray Detectors for Frame-Based Extra-Oral Dental Scanning

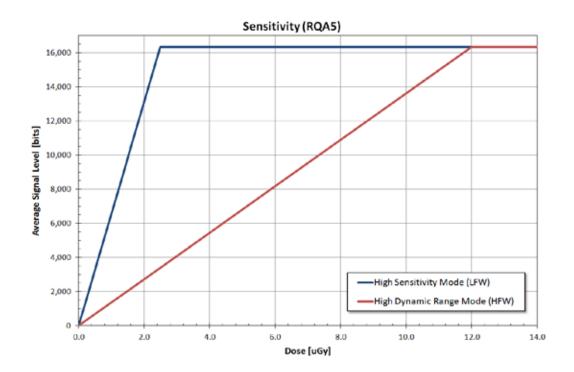

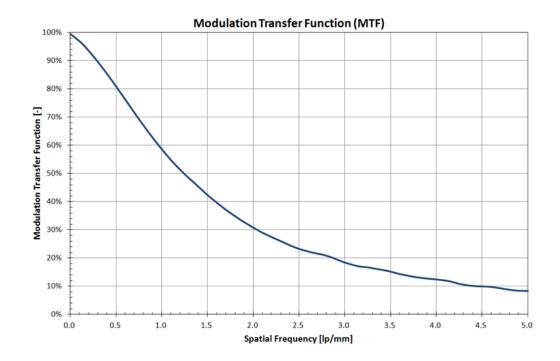

Xineos-1501/2301 CMOS X-ray Detectors for Frame-Based Extra-Oral Dental Scanning

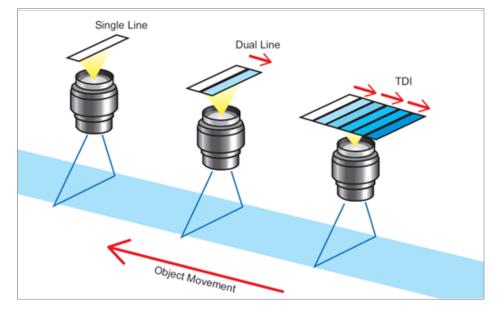


Digital Time Delayed Integration Mode (DTDI)


In Digital TDI mode, the Xineos-1501/2301 detectors capture and write full image frames to the internal memory, synchronized to the systems scanning motion, but outputs single line images to the system. Successive images are summed inside the detectors memory, while the frame memory is shifted after each line output. In this way, the effective integration time is extended while image sharpness is maintained.

Various line-scan technologies: single-line, dual-line and time delay integration (TDI). To achieve high responsivity, TDI uses multiple stages to capture multiple exposures. In these stages, photogenerated signal charges are transferred in sync with object motion. Dual-line scans are considered two-stage TDI's.


### 1. Capture individual frames (internal to detector)

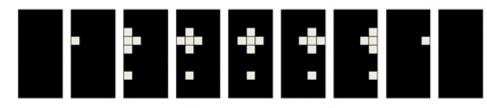

Time (& object motion)

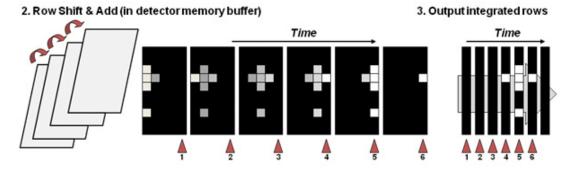

Xineos-1501/2301 CMOS X-ray Detectors for Frame-Based Extra-Oral Dental Scanning

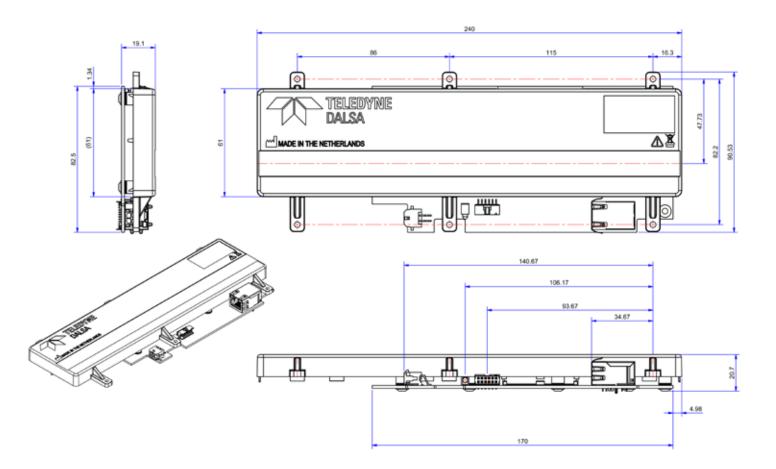

#### www.teledynedalsa.com

#### Americas

Waterloo, ON +1 519-886-6000 sales.sensors@teledynedalsa.com Europe Eindhoven, The Netherlands +31 40-259-9000 sales.sensors@teledynedalsa.com

### Asia Pacific

Tokyo, Japan +81 3-5960-6353 sales.sensors@teledynedalsa.com Shanghai, China +86 21-3368-0027 sales.sensors@teledynedalsa.com

Teledyne DALSA has its corporate offices in Waterloo, Canada

Teledyne DALSA reserves the right to make changes at any time without notice. Teledyne DALSA © 2018. 072315\_Xineos1501-2301\_datasheet.pdf

TELEDYNE DALSA Everywhere**you**look<sup>™</sup> Part of the Teledyne Imaging Group